




### Instructions for placement

Bucopreven is received in a transparent and closed bag. It should be washed before use. It has two elastic bands that hold it to the head, a short one that goes just behind the head (above the nape) and a long one that goes directly to the top of the head. It has a small plate (clip), made of aluminum, moldable on the nose (tailored to each user) which we will mold by lightly pressing the plate on both sides, with the masks on, of the nose.

### Instructions for removal

Before removal, hand washing should be performed again.

The rubber bands that hold the Bucopreven to the head from the back to the front should be removed, avoiding touching the textile part of it.

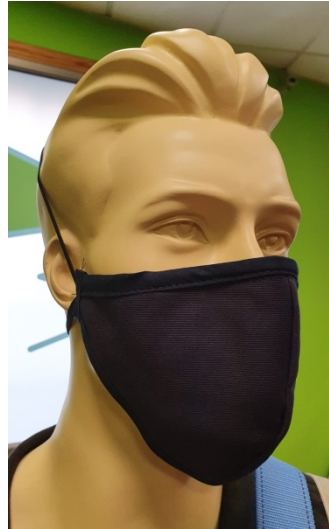

### Cleaning and storage instructions

Once the work day ends, we will proceed to clean it as follows:

We remove the mask following the previous instructions and we will put it under the tap with hot water, we will apply a detergent for the clothes that we have at home, we will rub both sides of it with foam, we will rinse with abundant and hot water, and without twisting the rod, we will drain the mask.

Immediately we will proceed to its drying, placing the mask in normal drying, lying or directly in the heat of a possible radiator, putting a towel and on top of the mask, avoiding possible sources of contamination while it dries.

It is also possible to clean it in the washing machine, using a short program at no more than 60°. It cannot be washed immediately, put it in a bag until it is washed, then discard the bag. The maximum number of washes is twenty-five (25).

Important! Be careful with the clip, it should not be closed completely when we do not have it on and it is recommended not to manipulate it except in the initial molding because the plate can break. You must wash your hands (soap and water and / or hydroalcoholic solution) before placing of the product

### Disposal instructions

Once the useful life of the hygienic mask has expired, it must be treated as household waste, throwing it into the container wrapped in a plastic bag, such as the one supplied with the product.